

reGO X-ray and ultrasound accessories

Kassettenschleusen



Cartridge Hatches

Der kürzeste Weg führt direkt durch die Wand. Deshalb bringt dieser Transport der Röntgenfilmkassetten im täglichen Routinebetrieb wirksame Arbeitserleichterung und spürbaren Zeitgewinn.

Die robuste Ganzstahlkonstruktion mit Spezialgrundierung hat einen einseitigen 2,0 mm starken Bleischutz. Der Einbau erfolgt wandbündig im Röntgenraum: Montagespannrahmen sowie Abdeckrahmen werden mitgeliefert.



Zweiweg-Durchreiche / 2-way type

The shortest way to transport the cassettes is directly through the wall. Consequently, the routine daily movement of the cassettes is more efficient and less time consuming. The strong construction of the cartridge hatch is completely made from steel, coated with a special primer. One side has a lead protection of 2,0 mm Pb. The cartridge hatch is intended for unilateral installation flush with the wall. The fitting clamp frame and two-part frame are part of the delivery.

Art. Nr. / art. no.	643080	643081
Modell / model	Einweg-Durchreiche / 2 Türen 1-way type / 2 doors	Zweiweg-Durchreiche / 4 Türen 2-way type / 4 doors
Maße / dimensions	28 x 54 x 55 cm (B x H x T) (w x h x d)	58 x 54 x 55 cm (B x H x T) (w x h x d)
Verriegelung bolt	mechanisch mechanical	mechanisch mechanical
Mauerdurchbruch (B x H) wall opening	28,5 x 54,5 cm (B x H) (w x h)	58,5 x 54,5 cm (B x H) (w x h)
Gewicht weight	24 kg	52 kg

Installation

Im Röntgenraum die Kassettenschleuse in die Maueröffnung einsetzen, horizontal ausrichten, mit Holzkeilen fixieren und festschrauben. Den umlaufenden Mauerspalt mit Gips o.ä. lichtdicht auffüllen. Abschließend auf der Dunkelkammerseite die Rahmenblenden anbringen.

Installation

Insert the cassette transfer cabinet into the wall opening from the side of X-ray room. Adjust the cabinet horizontally, fix it with wooden wedges properly and fix it then with screws. Fill the opening around the cassette transfer cabinet lightproof with gips. Apply the metal 4 pieces frame on the darkroom side.

REGO X-ray GmbH
Stuttgarter Str. 5
D-86154 Augsburg
Tel.: 0049/821/450435-0
Fax: 0049/821/450435-11
e-mail: info@rego.de
www.rego.de

